

Fitting the grab handle.



The grab handle will be fitted to the two slats pointed out, both on the back section, six slats apart.



This is the ideal location allowing easy access in to the bed and the perfect position to assist.



The first step is to screw the two battens supplied, to the underside of each of the two slats.



Grip the batten against the slat approximately 1cm from the rubber housing.



Screw through the slat into the batten approximately 5cm from each end of the batten.



Remove the two slats from their rubber housings by pulling back on the rubber housing.



At the same time, twist the slat and it will come out.



Pull the slat out from the housing on the opposite side.



Slip a jubilee clip over each end of the two slats.



Relocate the two slats.



Slip the grab handle through the Jubilee Clips



Push it up to the bed allowing space for bedding as required.



Lift the head end of the mechanism.



Step inside the back of the base.



Tighten the four Jubilee clips.



Check you have allowed a suitable distance from the edge of the mattress.



Replace the mattress and try it out.



Easy to fit and very rigid.